



ResMed

AirFit™ P30i

Fitting guide for AirFit P30i



- Pull the frame and headgear over the head until the elbow is positioned on top and the headgear is behind the head.



- Place the cushion under the nose and align the pillows into the nostrils.



- Tighten or loosen the headgear until the cushion fits comfortably under the nose.



- Squeeze the side buttons on the elbow and detach from the frame. Connect the air tubing from the device to the elbow, then reattach to the frame.

Additional options

For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

1 ResMed guided external clinical study of 23 current nasal pillows mask users from May to June, 2018 in Sydney, Australia who trialed ResMed AirFit P30i and Phillips™ Respironics DreamWear gel pillows. One participant withdrew consent after study completion (n=22).

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit is a trademark and/or registered trademark of the ResMed family of companies. DreamWear and Phillips are trademarks of Koninklijke Philips N.V, registered in the US and other countries. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2019 ResMed. 10111097/1 2019-01

[ResMed.com/AirFitP30i](https://www.resmed.com/AirFitP30i)